


Cleaning your TJF-Q180V

The TJF-Q180V has a number of features enabling easier reprocessing. This quick reference guide provides an overview of the main improvements to the pre-cleaning procedure.

CAUTION: This guide is only a summary of some key considerations to properly clean your duodenscope. Be sure to follow the detailed steps outlined in the TJF-Q180V REPROCESSING MANUAL that was included with your duodenscope purchase.


Pre-cleaning at bedside:

- The sealed elevator wire channel of the TJF-Q180V means that the elevator wire channel cannot be (and does not need to be) flushed or brushed.

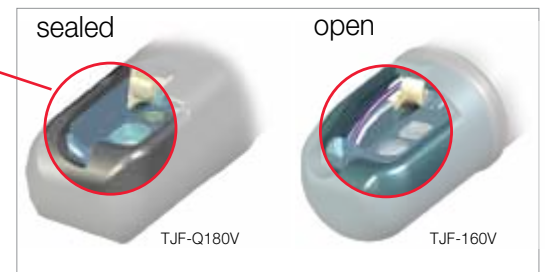

During manual cleaning:

- Use the recommended brushes to brush the front and rear side of the forceps elevator.
- The sealed elevator wire channel of the TJF-Q180V means that the elevator wire channel does not need to be (and cannot be) flushed and rinsed.
- Brush the forceps elevator including grooves and recess, then raise and brush the back and sides of the forceps elevator. Inspect whether there is debris on the forceps elevator and in the forceps elevator recess. Repeat until no visible debris remains.
- Operate the elevator control lever to raise and lower the forceps elevator in the detergent solution at least three times. Raise the forceps elevator and flush the interior of the forceps elevator recess with 30 mL of the detergent solution.

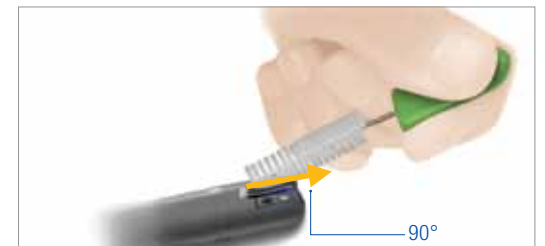

Prior to disinfection

- Set the forceps elevator to the intermediate position (approximately 45°) prior to placement into an automated endoscope reprocessor to enable sufficient cleaning and disinfection of the forceps elevator and recess.

