

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®/nStream®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite an EasySuite 4K, nCare GX/MX/RX, nStream G3, EasyLink Router, Thin Client, LiveStream StatusBoard Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites, nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: September 11, 2019

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

September 2019 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB4516655	nStream	2019-09 Servicing Stack Update for Windows 7 for x86-based Systems
KB4516065	nStream	2019-09 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB4474419	nStream	2019-09 Security Update for Windows 7 for x86-based Systems
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB4516655	VaultStream 2008	2019-09 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems
KB4516065	VaultStream 2008	2019-09 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4474419	VaultStream 2008	2019-09 Security Update for Windows Server 2008 R2 for x64-based Systems
KB4514602	VaultStream 2008	2019-09 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
VaultStream v10.x Updates – Windows® Server 2012		
KB4512938	VaultStream 2012	2019-09 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB4514604	VaultStream 2012	2019-09 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4516115	VaultStream 2012	2019-09 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB4516067	VaultStream 2012	2019-09 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB3013769	VaultStream 2012	Update for Windows Server 2012 R2
KB3013816	VaultStream 2012	Update for Windows Server 2012 R2
KB3172729	VaultStream 2012	Security Update for Windows Server 2012 R2
KB4087514	VaultStream 2012	Microsoft .NET Framework 4.8 Language Packs for Windows 8.1 and Server 2012 R2 for x64

VaultStream v10.x Updates – Windows® Server 2016		
KB4516044	VaultStream 2016	2019-09 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4512574	VaultStream 2016	2019-09 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4512495	VaultStream 2016	2019-08 Cumulative Update for Windows Server 2016 for x64-based Systems
EasyLink Router v8.0 Updates		
None		
nCare RX v8.0 and v8.1, nCare MX/ nStream GX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates		
KB4517297	nCare-SBE	2019-08 Update for Windows Embedded Standard 7 for x64-based Systems
KB4512193	nCare-SBE	2019-08 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4507004-x64_a7d988d65422c8cfbbed141c95aeea37a8743167.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4514602	nCare-SBE	2019-09 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4507004-x64_a7d988d65422c8cfbbed141c95aeea37a8743167.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4516065	nCare-SBE	2019-09 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4516655	nCare-SBE	2019-09 Servicing Stack Update for Windows Embedded Standard 7 for x64-based Systems
KB4516033	nCare-SBE	2019-09 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4474419	nCare-SBE	2019-09 Security Update for Windows Embedded Standard 7 for x64-based Systems
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4517297	Thin Client v8.0-ELR v8.1	2019-08 Update for Windows Embedded Standard 7 for x86-based Systems
KB4516065	Thin Client v8.0-ELR v8.1	2019-09 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4516655	Thin Client v8.0-ELR v8.1	2019-09 Servicing Stack Update for Windows Embedded Standard 7 for x86-based Systems
KB4516033	Thin Client v8.0-ELR v8.1	2019-09 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4474419	Thin Client v8.0-ELR v8.1	2019-09 Security Update for Windows Embedded Standard 7 for x86-based Systems
nCare MX v10.2.1		
KB4512574	nCare MX	2019-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4516044	nCare MX	2019-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4516115	nCare MX	2019-09 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

KB4524152	nCare MX	2019-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
KB4516061	All-In-One Client	2019-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4516115	MedPresence SA, and All-In-One Client	2019-09 Security Update for Adobe Flash Player for Windows 10 Version 1607
KB4516044	MedPresence SA, and All-In-One Client	2019-09 Cumulative Update for Windows 10 Version 1607 for x86-based Systems
KB4512574	MedPresence SA, and All-In-One Client	2019-09 Servicing Stack Update for Windows 10 Version 1607 for x86-based Systems
EasySuite 4K v10.1.2 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4512574	EasySuite 4K Controller	2019-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4516044	EasySuite 4K Controller	2019-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4516115	EasySuite 4K Controller	2019-09 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.1.2 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4524152	EasySuite 4K Video Manager	Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4516115	EasySuite 4K Video Manager	Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4512574	EasySuite 4K Video Manager	Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.0.x – LiveStream StatusBoard 2		
KB4516115	LSSB2	Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4516044	LSSB2	Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4512574	LSSB2	Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4524152	LSSB2	Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.32		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas