Ultravision™ Visual Field Clearing System

Ionwand™ Sterile Pack

- DE Ionwand™ sterile packung
- FR Paquet stérile Ionwand™
- ES Paquete estéril de Ionwand™
- IT Pacco Sterile Ionwand™
- NL Ionwand™ steriel pakket
- PT Embalagem Estéril Ionwand™
- SV Ionwand™ Sterilt Set
- EL Αποστειρωμένη συσκευασία Ionwand™
- DA Ionwand™ sterilpakke
- FI Ionwand™ - steriilipakkaus
- PL Zestaw sterylony Ionwand™
- LT Ionwand™ sterili pakuotė
- SK Sterilné balenie Ionwand™


Instructions, Gebrauchsanweisung, Consignes, Instrucciones, Istruzioni,
Instructies, Instruções, Bruksanvisningar, Οδηγίες, Vejledninger, Ohjeet,
Instrukcje, Instrukcija, Pokyny

Please read all information carefully before use.


Failure to do so may lead to serious injury to patient and operator.

Rx
ONLY


1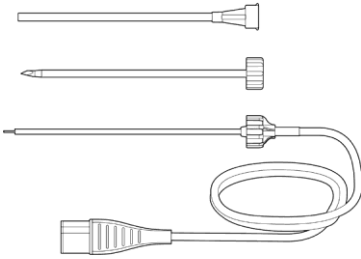


2

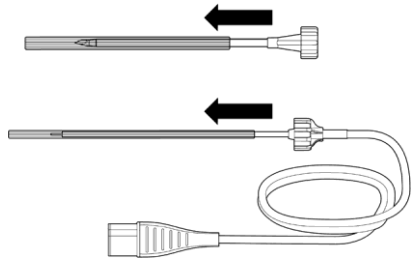
3

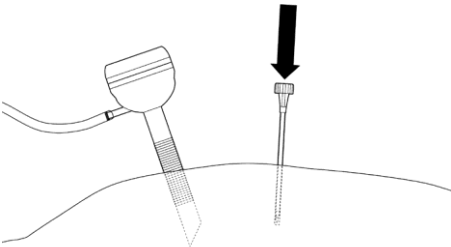
4

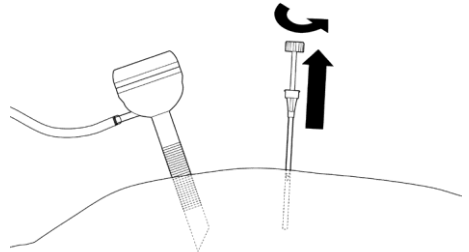5

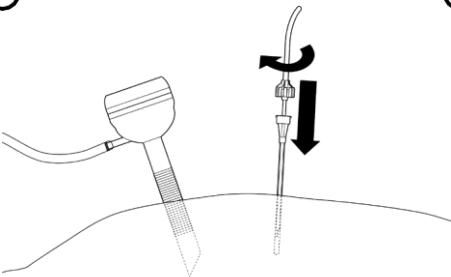6

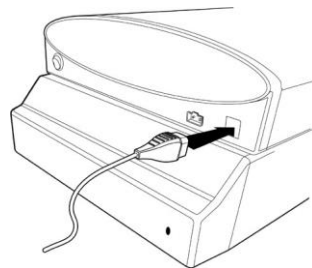Important

This booklet is designed to assist in using this product. It is not a reference to surgical techniques.

Device description

The Ionwand™ Sterile Pack is intended for use with the Ultravision™ System. The pack contains a sterile, single use device consisting of an Ionwand cable, a catheter and a trocar. The catheter and trocar are supplied pre-assembled. The trocar is used to introduce the catheter through the abdominal wall. After removal of the trocar the Ionwand cable is then inserted into the catheter and luer-locked in place in order to maintain an airtight seal.

Contents (Figure 1)

1. Ionwand cable
2. Catheter
3. Trocar

Indications

The Ultravision System is indicated for the clearance of smoke and other particulate matter that is created during laparoscopic surgery.

Contraindications

None.

Warnings

- Ultravision is not intended to be used in non-laparoscopic surgery procedures.
- Use Ultravision with caution in the presence of internal or external pacemakers. Similar to electrosurgical devices, Ultravision can affect the performance of a pacemaker. Consult the pacemaker manufacturer or hospital Cardiology Department for further information when use of Ultravision is planned in patients with cardiac pacemakers.
- Similar to electrosurgical devices, Ultravision may also interfere with the performance of an internal cardiac defibrillator (ICD). If the patient has an ICD contact the ICD manufacturer for instructions before performing an electrosurgical procedure with Ultravision.
- The Ultravision System should only be used by appropriately trained medical personnel.
- Single use only – reuse of this device poses a severe risk of cross-infection.
- Never re-use any sterile pack component. These have not been designed for re-sterilization.
- The Ionwand Sterile Pack has been designed specifically for use with the Ultravision System. Only use items supplied by Alesi Surgical Limited.
- If the pack is open or damaged, do not use and discard safely.
- If there is evidence of damage to any of the components, do not use and discard safely.
- Always insert the catheter and trocar under direct vision once the abdominal cavity has been insufflated.
- Avoid major arteries and veins when inserting the catheter.
- Never attempt to re-insert a partially or fully withdrawn trocar into the abdomen.
- Always dispose of trocar safely.
- Always insert the Ionwand cable into the patient before switching on the Ultravision Generator.
- As with electrosurgical instruments, contact between the Ionwand and uninsulated metallic instruments should be minimised to prevent electrostatic discharge to the operator or patient that may also transiently interfere with an ECG signal. Repeated contact of the Ionwand with a video laparoscope may lead to temporary interference with the visual display and/or damage to the laparoscope.
- Always switch off the Ultravision Generator before removing the Ionwand cable from the patient.

Before use

For detailed instructions on how to set up the Ultravision System for use with monopolar, bipolar or ultrasonic instruments, consult the Ultravision System User Manual.

Insertion of lonwand™

1. Open the lonwand Sterile Pack and remove the catheter/trocar assembly and the lonwand cable from the packaging.
2. Remove the protective sheaths from the trocar/catheter and lonwand cable and check the integrity of the components (Figure 2).
3. Introduce the catheter by positioning at 90° to the abdominal wall and piercing with the catheter/trocar assembly (Figure 3):
 - a. Always insert after the abdomen has been insufflated.
 - b. Always insert using direct vision and avoid major arteries or veins.
4. Remove the trocar from the catheter by holding the catheter and rotating the trocar anti-clockwise and dispose of safely (Figure 4).
5. Carefully insert the lonwand cable down the catheter until it protrudes from the tip and can be visualised internally within the patient with the laparoscopic camera. Secure the lonwand cable in place by holding the catheter and rotating the hub of the lonwand cable clockwise until it tightens (Figure 5).
6. Adjust the height of the catheter so that the lonwand cable is free from contact with patient tissue and surgical instruments.
7. Connect the lonwand connector to the active socket on the face of the Ultravision generator (Figure 6).

During surgical procedure

8. Ensure that the lonwand cable is free of surgical debris and not touching patient tissue or a surgical instrument.
9. Switch on the Ultravision Generator. If the battery status light indicator is flashing red and the battery alarm is sounding, replace the battery with a fully charged unit.
10. Perform surgical procedure.
11. In the event that the proximity indicator sounds when the lonwand cable is not touching patient tissue or a surgical instrument, perform the following steps:
 - a. Switch off the Ultravision Generator.
 - b. Remove the lonwand cable from the catheter by holding the catheter and twisting the lonwand cable anti-clockwise, ensuring the catheter is not displaced.
 - c. Clean the lonwand cable and brush filaments by enclosing it within a sterile swab moistened with sterile saline and gently wiping it, moving the swab from the insulated portion of the lonwand cable towards the metal brush. Repeat until all contamination has been removed and the brush is clean of contaminants and any swab fibres.
 - d. Re-introduce the lonwand cable into the catheter and secure by holding the catheter and twisting the lonwand cable hub in a clockwise direction.
 - e. Switch on the Ultravision Generator and confirm that the indicator condition has resolved and that smoke clearing has been resumed.
 - f. If the alarm continues to sound, discard the lonwand cable and replace.
12. Switch off Ultravision Generator once surgical cutting is complete before removing from patient.

End of surgical procedure

13. Ensure that the Ultravision Generator is OFF.
14. Disconnect the lonwand cable from the generator.
15. Carefully remove the lonwand cable from the catheter.
16. Remove the catheter and apply dressing to the wound site.
17. Dispose of lonwand cable and catheter safely.

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© Alesi Surgical Limited 2020
Advena Ltd., Tower Business Centre