



## Microsoft Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

**Product:** nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

**Date:** April 16, 2018

**Critical Security Bulletins (Obtained from Microsoft):**

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

**Approved Updates:**

April 2018 Updates		
KB Number	Platform	Title
KB2952664	nStream	Update for Windows 7
KB4093118	nStream	2018-04 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStream	Windows Malicious Software Removal Tool - April 2018
KB4093118	VaultStream	2018-04 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - April 2018
KB4091756	ELR v8.0	2018-04 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4093223	ELR v8.0	2018-04 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4093224	ELR v8.0	2018-04 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4093257	ELR v8.0	2018-04 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4101864	ELR v8.0	2018-04 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4088878	nCareRX/MX/GX-SBE	2018-03 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4088875	nCareRX-SBE	
KB4093108	nCareRX/MX/GX-SBE	2018-04 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4093118	nCareRX/MX/GX-SBE	2018-04 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4099467	nCareRX/MX/GX-SBE	2018-03 Update for Windows Embedded Standard 7 for x64-based Systems
KB4099950	nCareRX/MX/GX-SBE	2018-04 Update for Windows Embedded Standard 7 for x64-based Systems

		Systems
KB4088878	ThinClient-ELR v8.1	2018-03 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4088875	ELR v8.1	
KB4093108	ThinClient-ELR v8.1	2018-04 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4093118	ELR v8.1	2018-04 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
		pciclearstalecache_f0e916619e5084d65a0df11a88f52a583e984988.exe
KB4099467	ThinClient-ELR v8.1	2018-03 Update for Windows Embedded Standard 7 for x86-based Systems
KB4099950	ThinClient-ELR v8.1	2018-04 Update for Windows Embedded Standard 7 for x86-based Systems

**PHP Updates**

**5.3.36**

**For further questions concerning this notification please call Technical Support at (866) 376-6800.**