

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®/nStream®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite an EasySuite 4K, nCare GX/MX/RX, nStream G3, EasyLink Router, Thin Client, LiveStream StatusBoard Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites, nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: August 14, 2019

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

August 2019 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB890830	nStream	Windows Malicious Software Removal Tool - August 2019
KB4474419	nStream	2019-08 Security Update for Windows 7 for x86-based Systems
KB4512506	nStream	2019-08 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB4512506	VaultStream 2008	2019-08 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB915597	VaultStream 2008	Security Intelligence Update for Windows Defender Antivirus - (Version 1.299.2006.0)
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - August 2019
KB4474419	VaultStream 2008	2019-08 Security Update for Windows Server 2008 R2 for x64-based Systems
KB4503548	VaultStream 2008	Microsoft .NET Framework 4.8 for Windows Server 2008 R2 for x64
VaultStream v10.x Updates – Windows® Server 2012		
KB3162835	VaultStream 2012	Update for Windows Server 2012 R2
KB3118401	VaultStream 2012	Update for Windows Server 2012 R2
KB3012997	VaultStream 2012	Italian Language Pack - Windows Server 2012 R2 - [it-IT_LP]
KB2978122	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3146723	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3037576	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3027209	VaultStream 2012	Update for Windows Server 2012 R2
KB3081320	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3012235	VaultStream 2012	Update for Windows Server 2012 R2
KB2989930	VaultStream 2012	Update for Windows Server 2012 R2
KB3127222	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64

KB3074545	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3121261	VaultStream 2012	Update for Windows Server 2012 R2
KB2966828	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3044374	VaultStream 2012	Update for Windows Server 2012 R2
KB3054256	VaultStream 2012	Update for Windows Server 2012 R2
KB3126593	VaultStream 2012	Security Update for Windows Server 2012 R2
KB2973114	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3000483	VaultStream 2012	Security Update for Windows Server 2012 R2
KB2894852	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3013410	VaultStream 2012	Update for Windows Server 2012 R2
KB3122651	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64
KB4507448	VaultStream 2012	2019-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB3134179	VaultStream 2012	Update for Windows Server 2012 R2
KB3185319	VaultStream 2012	Cumulative Security Update for Internet Explorer 11 for Windows Server 2012 R2
KB2966826	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3077715	VaultStream 2012	Update for Windows Server 2012 R2
KB3147071	VaultStream 2012	Update for Windows Server 2012 R2
KB2967917	VaultStream 2012	Update for Windows Server 2012 R2
KB3149157	VaultStream 2012	Update for Windows Server 2012 R2
KB3045999	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3132080	VaultStream 2012	Update for Windows Server 2012 R2
KB4033428	VaultStream 2012	Update for Windows Server 2012 R2
KB3012702	VaultStream 2012	Update for Windows Server 2012 R2
KB2919355	VaultStream 2012	Windows Server 2012 R2 Update
KB2972103	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3097992	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3179574	VaultStream 2012	Update for Windows Server 2012 R2
KB3046737	VaultStream 2012	Update for Windows Server 2012 R2
KB3023219	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3115224	VaultStream 2012	Update for Windows Server 2012 R2
KB3088195	VaultStream 2012	Security Update for Windows Server 2012 R2
KB4054534	VaultStream 2012	Microsoft .NET Framework 4.7.2 Language Packs for Windows Server 2012 R2 for x64
KB3042058	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3123245	VaultStream 2012	Update for Windows Server 2012 R2
KB3175024	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3071756	VaultStream 2012	Security Update for Windows Server 2012 R2
KB2972213	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3172614	VaultStream 2012	Update for Windows Server 2012 R2
KB3013816	VaultStream 2012	Update for Windows Server 2012 R2



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

KB3013769	VaultStream 2012	Update for Windows Server 2012 R2
KB4507463	VaultStream 2012	2019-07 Preview of Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4486105	VaultStream 2012	Microsoft .NET Framework 4.8 for Windows Server 2012 R2 for x64
KB4512488	VaultStream 2012	2019-08 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - August 2019
KB4507422	VaultStream 2012	2019-07 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4087514	VaultStream 2012	Microsoft .NET Framework 4.8 Language Packs for Windows 8.1 and Server 2012 R2 for x64
VaultStream v10.x Updates – Windows® Server 2016		
KB4507459	VaultStream 2016	2019-07 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4507460	VaultStream 2016	2019-07 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4509091	VaultStream 2016	2019-07 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4512517	VaultStream 2016	2019-08 Cumulative Update for Windows Server 2016 for x64-based Systems
KB3193497	VaultStream 2016	Polish Language Pack - Windows Server 2016 for AMD64-based Systems - [pl-PL LP]
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - August 2019
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - July 2019
EasyLink Router v8.0 Updates		
None		
nCare RX v8.0 and v8.1, nCare MX/ nStream GX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates		
KB4512506	nCare-SBE	2019-08 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4512486	nCare-SBE	2019-08 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4474419	nCare-SBE	2019-08 Security Update for Windows Embedded Standard 7 for x64-based Systems
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4512506	Thin Client v8.0-ELR v8.1	2019-08 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4512486	Thin Client v8.0-ELR v8.1	2019-08 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4474419	Thin Client v8.0-ELR v8.1	2019-08 Security Update for Windows Embedded Standard 7 for x86-based Systems
nCare MX v10.2.1		
KB4512517	nCare MX	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4512495	nCare MX	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
None		
EasySuite 4K v10.1.2 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4512495	EasySuite 4K Controller	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

KB4512517	EasySuite 4K Controller	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.1.2 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4512517	EasySuite 4K Video Manager	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4512495	EasySuite 4K Video Manager	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.0.x – LiveStream StatusBoard 2		
KB4512517	LSSB2	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4512495	LSSB2	2019-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.31		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas