

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: March 11, 2020

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

March 2020 Updates		
KB Number	Platform	Title
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - March 2020
VaultStream v10.x Updates – Windows® Server 2012		
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - March 2020
KB4541509	VaultStream 2012	2020-03 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4540725	VaultStream 2012	2020-03 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB4538158	VaultStream 2012	2020-02 Preview of Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
VaultStream v10.x Updates – Windows® Server 2016		
KB4540670	VaultStream 2016	2020-03 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - March 2020
KB4052623	VaultStream 2016	Update for Windows Defender Antivirus antimalware platform - (Version 4.18.2001.10)
KB4540723	VaultStream 2016	2020-03 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4537477	VaultStream 2016	2020-02 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
nCare MX v10.2.2		
KB4540670	nCare MX	2020-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4540723	nCare MX	2020-03 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4052623	nCare MX	Update for Windows Defender Antivirus antimalware platform (Version 4.18.2001.10)

EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4540670	EasySuite 4K Controller	2020-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4540723	EasySuite 4K Controller	2020-03 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4540670	nCare RX2	2020-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4540723	nCare RX2	2020-03 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4540723	LiveStream StatusBoard 2	2020-03 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4540670	LiveStream StatusBoard 2	2020-03 Cumulative Update for Windows Server 2016 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.15 available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas