



Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., EasySuite™, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and ES4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: April 14, 2021

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

April 2021 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB5000848	VaultStream 2012	2021-03 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB3053863	VaultStream 2012	Update for Windows Server 2012 R2
KB5001382	VaultStream 2012	2021-04 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.88
KB5001403	VaultStream 2012	2021-04 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB4577586	VaultStream 2012	Update for Removal of Adobe Flash Player for Windows Server 2012 R2 for x64-based systems
VaultStream v10.x Updates – Windows® Server 2016		
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.87
KB4589210	VaultStream 2016	2021-01 Update for Windows Server 2016 for x64-based Systems
KB5000803	VaultStream 2016	2021-03 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5001347	VaultStream 2016	2021-04 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.88
KB5001402	VaultStream 2016	2021-04 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB5001078	VaultStream 2016	2021-02 Servicing Stack Update for Windows Server 2016 for x64-based Systems
nCare MX v10.2.3		
KB4589210	nCare MX	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5000803	nCare MX	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4577586	nCare MX	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems

KB5001402	nCare MX	2021-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5001347	nCare MX	2021-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4589210	EasySuite 4K Controller	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5000803	EasySuite 4K Controller	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4577586	EasySuite 4K Controller	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5001347	EasySuite 4K Controller	2021-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001402	EasySuite 4K Controller	2021-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4589210	EasySuite 4K Video Manager	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5000803	EasySuite 4K Video Manager	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4577586	EasySuite 4K Video Manager	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5001347	EasySuite 4K Video Manager	2021-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001402	EasySuite 4K Video Manager	2021-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4589210	LiveStream StatusBoard 2	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5000803	LiveStream StatusBoard 2	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4577586	LiveStream StatusBoard 2	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5001347	LiveStream StatusBoard 2	2021-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001402	LiveStream StatusBoard 2	2021-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
MedPresence Tower v10.3		
KB5001402	MedPresence Tower	2021-04 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5001347	MedPresence Tower	2021-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.27) available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024