

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., EasySuite™, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: February 10, 2021 and March 10, 2021

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

February and March 2021 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.86
KB4601384	VaultStream 2012	2021-02 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4603004	VaultStream 2012	2021-02 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB5000848	VaultStream 2012	2021-03 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.87
VaultStream v10.x Updates – Windows® Server 2016		
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.86
KB4601051	VaultStream 2016	2021-02 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB4601392	VaultStream 2016	2021-02 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4601318	VaultStream 2016	2021-02 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5001078	VaultStream 2016	2021-02 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.87
KB4589210	VaultStream 2016	2021-01 Update for Windows Server 2016 for x64-based Systems
KB5000803	VaultStream 2016	2021-03 Cumulative Update for Windows Server 2016 for x64-based Systems
nCare MX v10.2.3		
KB4577586	nCare MX	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5000803	nCare MX	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4589210	nCare MX	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5001078	nCare MX	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4601318	nCare MX	2021-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems

KB4601051	nCare MX	2021-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4601392	nCare MX	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4577586	EasySuite 4K Controller	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5000803	EasySuite 4K Controller	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4589210	EasySuite 4K Controller	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5001078	EasySuite 4K Controller	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4601318	EasySuite 4K Controller	2021-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4601051	EasySuite 4K Controller	2021-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4601392	EasySuite 4K Controller	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4577586	EasySuite 4K Video Manager	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
KB5000803	EasySuite 4K Video Manager	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4589210	EasySuite 4K Video Manager	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB5001078	EasySuite 4K Video Manager	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4601318	EasySuite 4K Video Manager	2021-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4601051	EasySuite 4K Video Manager	2021-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4601392	EasySuite 4K Video Manager	2021-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4589208	LiveStream StatusBoard 2	2021-01 Update for Windows 10 Version 1809 for x64-based Systems
KB5000822	LiveStream StatusBoard 2	2021-03 Cumulative Update for Windows 210 Version 1809 for x64-based Systems
KB5000859	LiveStream StatusBoard 2	2021-03 Servicing Stack Update for Windows 10 Version 1809 for x64-based Systems
KB4601383	LiveStream StatusBoard 2	2021-02-Cumulative Update Preview for Windows 10 version 1809 for x64-based Systems
KB4577586	LiveStream StatusBoard 2	Update for Removal of Adobe Flash Player for Windows 10 Version 1809 for x64-based systems
KB4602298	LiveStream StatusBoard 2	2021-02 Cumulative Update Preview for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64
KB4601345	LiveStream StatusBoard 2	2021-02 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB4598296	LiveStream StatusBoard 2	2021-01 Cumulative Update Preview for Windows 10 Version 1809 for x64-based Systems
KB4598461	LiveStream StatusBoard 2	2021-01 Cumulative Update Preview for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64

KB4535680	LiveStream StatusBoard 2	Security Update for Windows 10 Version 1809 for x64-based Systems
KB4598230	LiveStream StatusBoard 2	2021-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
MedPresence Tower v10.3		
KB4601318	MedPresence Tower	2021-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4535680	MedPresence Tower	Security Update for Windows 10 Version 1607 for x64-based Systems
KB5001078	MedPresence Tower	2021-02 Service Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4601051	MedPresence Tower	2021-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB890830	MedPresence Tower	Windows malicious Software Removal Tool x64 – v5.86
KB4598243	MedPresence Tower	2021-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5000080 3	MedPresence Tower	2021-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4589210	MedPresence Tower	2021-01 Update for Windows 10 Version 1607 for x64-based Systems
KB4577586	MedPresence Tower	Update for Removal of Adobe Flash Player for Windows 10 Version 1607 for x64-based systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.27 available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024