

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., EasySuite™, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: May 12, 2021

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

May 2021 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB5003209	VaultStream 2012	2021-05 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB5001881	VaultStream 2012	2021-05 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.89
KB5003209	VaultStream 2012	2021-05 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB5001881	VaultStream 2012	2021-05 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
VaultStream v10.x Updates – Windows® Server 2016		
KB5001841	VaultStream 2016	2021-05 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB5003197	VaultStream 2016	2021-05 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.89
	VaultStream 2016	VMware, Inc. - Display - 8.17.2.14
nCare MX v10.2.3		
KB5003197	nCare MX	2021-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001841	nCare MX	2021-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB5003197	EasySuite 4K Controller	2021-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001841	EasySuite 4K Controller	2021-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		

KB5003197	EasySuite 4K Video Manager	2021-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5001841	EasySuite 4K Video Manager	2021-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB5001841	LiveStream StatusBoard 2	2021-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5003197	LiveStream StatusBoard 2	2021-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
MedPresence Tower v10.3		
KB5001841	MedPresence Tower	2021-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5003197	MedPresence Tower	2021-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.27) available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024