

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: October 14, 2020

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

October 2020 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB4580347	VaultStream 2012	2020-10 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4580325	VaultStream 2012	2020-10 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB4579979	VaultStream 2012	2020-10 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
VaultStream v10.x Updates – Windows® Server 2016		
KB4580346	VaultStream 2016	2020-10 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4578969	VaultStream 2016	2020-10 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
nCare MX v10.2.2		
KB4580346	nCare MX	2020-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4580325	nCare MX	2020-10 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4578969	nCare MX	2020-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4580346	EasySuite 4K Controller	2020-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4580325	EasySuite 4K Controller	2020-10 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4578969	EasySuite 4K Controller	2020-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4580346	EasySuite 4K Video Manager	2020-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems

KB4580325	EasySuite 4K Video Manager	2020-10 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4578969	EasySuite 4K Video Manager	2020-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4580346	LiveStream StatusBoard 2	2020-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4580325	LiveStream StatusBoard 2	2020-10 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4578969	LiveStream StatusBoard 2	2020-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
MedPresence Tower v10.3		
KB4580346	MedPresence Tower	2020-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4580325	MedPresence Tower	2020-10 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4578969	MedPresence Tower	2020-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
PHP Updates		
5.6.40 and 7.1.33 (7.3.23 available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas